



CLICK-IT PUTS YOU IN
TOTAL CONTROL

The aesthetic, self-ligating bracket system that delivers non-stop performance in every phase.

©
T
E
C
N
I
C
L
I
C
I
T

WHY CLICK-IT IS DIFFERENT

Click-It is the premier aesthetic, self-ligating bracket system with a true start to finish design you and your patients will appreciate. Welcome to a new level of treatment satisfaction and efficiency.

01 Total Control, Every Phase

The unique, four-walled design enables optimum archwire-to-slot interaction. Click-It adjusts to accommodate the mode of interaction from the start of the treatment to an effortless finish. Your patients can be confident treatment will go as planned.



02 Quick, Simple Wire Changes

Opens easily with forceps that apply no unilateral force. Closes with light, fingertip pressure. Less chair time for you and less inconvenience for your patients.





03 Breakthrough Aesthetic Design

Complete ceramic face without any visible metal parts or clips. With exclusive *Personalized Color-Matching Technology*[®], you can assure patients they'll look great during and after treatment.



04 Clean, Safe Debonding

Simplified debonding using an exclusive polymer mesh base – preventing direct contact between ceramic and tooth enamel to assure patient safety and comfort.



THE SELF-LIGATING BRACKET THAT GIVES YOU

CONTROL AT EVE

Click-It uses a unique, four-walled archwire slot to adjust to varying treatment objectives perfect for any patient, ensuring exceptional performance in every phase of treatment.



PHASE 1



FIGURE 1



FIGURE 2

Performs Faster

Click-It produces faster aligning and leveling due to a four-walled archwire slot.

- Bracket engages the wire passively with a width of 0.022"
- Slot enables sliding without undesirable, excessive torque
- Patient discomfort reduced with oversized slot-to-archwire clearance to soften the wire force (Figures 1 and 2)

RY PHASE



PHASE 2

Promotes More Movement

Click-It eases into progressive mode with an exclusive flat spring.

- Archwire slot quickly adjusts in size and applies a “soft” restriction on the wire to introduce progressive mode
- “Soft” interference provides controlled sliding for space closure and ideal parallel root movement as the wire size increases
- Controlled sliding enabled without any binding



PHASE 3

Delivers the Perfect Finish

Click-It provides comprehensive finishing and full prescription expression with active wire engagement.

- Simplified space closure, adjustment to midline and final positioning with use of sliding mechanics
- Total control for root paralleling and incisor torque requirements provided by full wire sizes in the 0.022” archwire slot

THE TWO-MINUTE WIRE CHANGE

IT'S ABOUT TIME

Easy Ligation Steps

Click-It features a spring-loaded jaw mechanism designed for smooth, efficient operation and patient comfort – with the option to use ligatures when necessary.



STEP 1 Align the archwire to the archwire slot with the bracket in the open position.



STEP 2 Apply light pressure across the wire at each end of the open bracket with your fingers.



STEP 3 Listen for the signature “click” to confirm proper wire placement and complete slot closure.

[VIEW VIDEO](#)





*Quick and easy wire changes
using forceps designed especially
for Click-It.*

Fast Wire Removal

Wire removal is as effortless as ligation with the efficient spring-and-cam technology. Using forceps, simply apply opposing bidirectional force on the bracket.



STEP Place forceps tips in the center of the bracket at the bottom of the jaw.

1

STEP Gently squeeze, steadily increasing pressure until the bracket clicks open.

2

STEP The spring system automatically ejects the wire out of the bracket.

3


[VIEW VIDEO](#)



BREAKTHROUGH AESTHETICS

TO DRIVE PATIENTS TO YOUR PRACTICE

More patients want aesthetic treatment solutions. Make their decision simple with breakthrough aesthetics and a treatment experience only Click-It can offer.

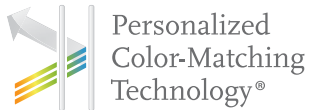

Click-It features the exclusive Personalized Color-Matching Technology for a look of total transparency. Brackets blend so they match your patients' teeth.

Unsurpassed aesthetics are further achieved with a 100% ceramic face, hidden tie-wings and no visible metal parts or clips.



*Click-It brackets displayed above on various tooth shades.
Click-It is the perfect match for any patient.*



DESIGN MAKES THE DIFFERENCE

See how Click-It stacks up against other brackets for results you and your patients can be confident in.



Benefits

	In-Ovation® C	Damon® Clear™	Damon® 3	Clarity™ SL	Click-It®
Truly aesthetic: no visible metallic parts		●			●
Active, progressive and passive ligation	●				●
Ease of use – closing mechanism activated by pressure on the wire				●	●
Complete control with four solid walls		●	●		●
Easy opening/closing even with the maximum wire size		●	●		●
Efficient movement with reduced friction at the contact points		●			●
Hygienic without food traps					●
Optimum appearance ensured with hidden tie wings					●
For your patient's comfort: no unidirectional forces for opening				●	●

YOUR PATIENTS ARE OUR PATIENTS

At TP Orthodontics we believe your patients are our patients. Click-It is designed with them in mind, combining breakthrough aesthetics with powerful technologies for a treatment experience no other bracket achieves.

clickit.tportho.com



The Sound of Certainty

Click-It gets its name from the clicking sound you hear when the archwire is properly seated and the bracket is locked in place.



For more information, visit
clickit.tportho.com or call
800-348-8856.



facebook.com/toprtho

youtube.com/tportho



TP Orthodontics, Inc.

Innovators of Personalized Color-Matching Technology®