Enamel Conditioner Solution

Series 150 - 151

TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672 USA

tportho.com

0120

TP Orthodontics Europe
Fountain Ct.
12 Bruntcliffe Way
Morley, Leeds LS27 0JG UK

© 2006 TP Orthodontics, Inc.

996-225, Rev. 1

ENGLISH

-  Clean teeth with plain pumice and water. Thoroughly clean teeth to be bonded with plain pumice (no oils or fluoride), using a rubber cup or bristle brush. Take care not to irritate the gingival tissue, as this may cause bleeding which could contaminate the bonding area.
-  Rinse, isolate and dry. Rinse thoroughly with water, isolate the teeth and air dry.
-  Apply Enamel Conditioner. Dispense one or two drops of Enamel Conditioner onto work pad and, using cotton forceps and one foam pellet applicator, dab conditioner onto teeth to be bonded. Condition for 40 seconds. It is only necessary to condition an area slightly larger than the actual bonding site. All teeth to be bonded in one arch may be conditioned simultaneously.
-  Rinse, isolate and dry. Flush teeth thoroughly with oil-free water to remove all traces of Enamel Conditioner and evacuate. Re-isolate and dry teeth to be bonded with oil- and water-free air. After drying, conditioned areas should appear chalky white. If not, re-condition teeth for approximately 10 more seconds. If a portion of a tooth exhibits a greenish hue, dental plaque is still present and the tooth must be re-pumiced and re-conditioned.

Storage: Best temperature for use is 68°-77°F (20°-25°C). Storage temperature is best at 35°-80°F (2°-27°C). Enamel Conditioner has a shelf life of 3 years.

CAUTION: Enamel Conditioner contains phosphoric acid. Avoid contact with skin, eyes, soft tissue, and dentin. In case of contact with skin or eyes, flush immediately with water, get medical attention for eyes. Do not take internally.

Note: This is a single-use device. Any reuse of this product will run the risk of cross-contamination to patients.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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