INSTRUCTIONS

Flip-Lock® HERBST® Appliance

Series 610

Flip-Lock is a registered trademark of TP Orthodontics, Inc. and manufactured under US Patent 5,620,321 and other patents pending. HERBST is a registered trademark of Dentsaur, Inc.

 TP Orthodontics, Inc.
100 Center Plaza
La Porte, Indiana 46350-9672 USA

tportho.com

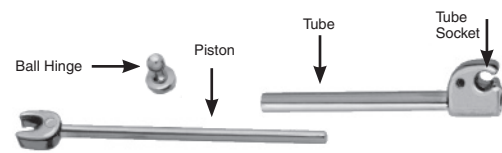
 0120 TP Orthodontics Europe
Fountain Ct.
12 Bruntcliffe Way
Morley, Leeds LS27 0JG UK

© 2006 TP Orthodontics, Inc.

996-816, Rev. 0

ENGLISH


The Flip-Lock HERBST is designed for ease of use. There's no need for special tools or complicated adjustments. Simply follow the steps listed below.

After cementation of the appliance, prepare tube and piston assembly for proper length. Tubes should be cut long enough to position the mandible in a Class I position. The pistons should extend no more than 2-3 mm beyond the distal end of the tubes at full closure. It is imperative that the pistons aren't so short that they separate from the tubes upon maximum opening (i.e., when the patient yawns).


- Slide a piston into a tube and hold in the correct orientation as shown, with the tube socket up. Note that the tube sockets are color coded:

**Left is red.
Right is green.**

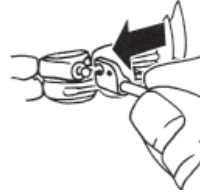
The tube socket can only be engaged on the maxillary distal ball hinge from one direction—the mesial. Slide the tube socket from mesial to distal end onto the maxillary ball hinge.

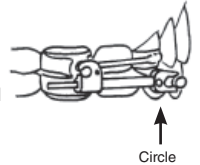
- The tube is not locked onto the maxillary ball hinge until the tube is laid parallel to buccal tooth surfaces. Before attaching the piston on the mandibular ball hinge, make sure the piston sockets also have a correct orientation. A small circle on the socket represents the labial surface. This circle should go toward the cheek, not toward the teeth.

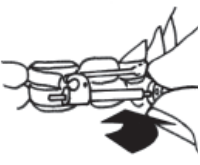- Slide the socket mesially over the mandibular ball hinge. The assembly will snap into place. Although not routinely necessary, you may gently crimp the piston socket onto the mandibular hinge to provide additional security.

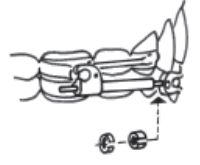- Activate the Flip-Lock HERBST periodically (usually at 6 to 8 week intervals) by adding TPO crimpable 1mm or 2mm spacers onto the exposed piston. Using crimpable spacers eliminates the need to remove the tubes and pistons. The mandible is initially advanced approximately 6-8mm, and then incrementally advanced until the desired occlusion is achieved. Average treatment time is 12 months.

NOTE: This is a single-use device. Any reuse of this product will run the risk of cross-contamination to patients.

Please read the following to the patient when placing the Flip-Lock HERBST Appliance:

The appliance is designed to withstand the normal forces of the mouth. The application of outside forces on these parts may cause them to become disconnected. Therefore, please do not apply forces to the parts in your mouth with your fingers or any other objects.

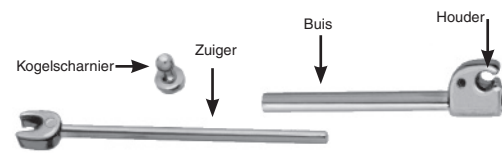
If any of the parts of this appliance become completely disconnected and loose in your mouth, immediately remove them from your mouth to avoid any possibility of swallowing the loose part or parts. Then immediately contact your orthodontist.

Warning: This stainless steel product contains nickel and chromium. Direct patient to consult physician if allergic reaction occurs.

NOTICE: TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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