## PORTUGUESE

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Order online at [tportho.com](http://tportho.com).

## INSTRUCTIONS

# TPO® Blue Etchant Gel

Series 150

 TP Orthodontics, Inc.  
100 Center Plaza  
La Porte, Indiana 46350-9672 USA

[tportho.com](http://tportho.com)

 0120  TP Orthodontics Europe  
Fountain Ct.  
12 Bruntcliffe Way  
Morley, Leeds LS27 0JG UK

## ENGLISH

-  1. Remove cap from syringe and insert a syringe tip.
-  2. Thoroughly clean teeth to be bonded with plain pumice (no oils or fluoride) using a rotating bristle brush or rubber cup. Do not irritate the gingival tissue, as this may cause slight bleeding which could contaminate the bonding area.
-  3. Rinse teeth thoroughly with water, isolate teeth and air dry.
-  4. Apply directly to each tooth. Condition for **30 seconds**. Condition an area only slightly larger than the actual bonding site. All teeth to be bonded in one arch may be conditioned simultaneously.
-  5. After use, remove syringe tip and replace cap. Disposable tips may be ordered in packs of 5, order number 150-714.
-  6. Flush teeth thoroughly with an oil-free air/water spray to remove all traces of conditioner and evacuate. Do not allow the patient to rinse or permit contact of the conditioned enamel with saliva or mucosal tissue. Re-isolate and dry teeth to be bonded with oil- and water-free air (preferably warm air). If conditioned areas do not appear chalky white, re-condition for approximately one more minute.

CAUTION: Contains 37% phosphoric acid. Avoid contact with eyes and skin. In case of contact, immediately wash with water and get medical attention for eyes. Do not take internally. Wash hands after use. Discontinue use if irritation occurs.

Best temperature for use is 68°-77°F (20°-25°C). Storage temperature is best at 35°-80°F (2°-27°C). Blue Etchant has a shelf life of 3 years.

### NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

Order online at [tportho.com](http://tportho.com).

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