**TP Orthodontics
Light Cure
Promoter
Instructions**

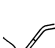
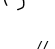
Use TP Orthodontics Light Cure Promoter with light cure adhesives to maximize bond strength on metal surfaces such as amalgam, stainless steel or gold. It will increase adhesion to composite or acrylic restorations, or to hypocalcified, fluorosed or deciduous enamel.

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Instructions

-  1. Thoroughly clean teeth with plain pumice using a rotating bristle brush or rubber cup.
-  2. Rinse teeth thoroughly with water, isolate teeth and air dry.
-  3. Apply Enamel Conditioner and condition for 40 seconds.
-  4. Rinse teeth thoroughly with water, isolate teeth and air dry. NOTE: If bonding to a metal surface, roughen with a medium diamond bur or micro etcher. When bonding to an acrylic or composite surface, lightly roughen the surface with a fine diamond bur.
-  5. Dispense a few drops of Promoter onto work pad. Using a disposable brush, apply 3-4 generous coats on enamel surface.
-  6. Thoroughly air dry after last coat of Promoter has been applied. The enamel will now appear shiny.
-  7. Dispense one or two drops of Light Cure Sealant onto work pad. Using a disposable brush, apply a thin, uniform coat of sealant onto the primed enamel.
-  8. Apply Light Cure Adhesive and light cure according to adhesive instructions.

Storage: Best temperature for use is 68°-77° F (20°-25° C). Storage temperature is best at 35°-80° F (2°-27° C). TPO Light Cure Promoter has a 3-year shelf life without refrigeration.

NOTICE:

TP Orthodontics, Inc. warrants that its products are free from defects in materials and workmanship. TPO's liability and the customer's sole remedy in the event of any claimed defect shall be limited to replacement of the product, or at TPO's discretion, refund of the purchase price. This warranty is in lieu of any other warranties, express or implied, including any implied warranty of merchantability, fitness for a particular purpose, or otherwise, and of any other obligations or liability on TPO's part. Other than the warranties listed above, under no circumstances will TPO be liable for any loss, damages arising in connection with use, or inability to use, TPO's products.

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